LH-Ovulationstest Mittelstrahl, 25mIU/ml

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REFERENZEN

[1] Sonya Godbert, Fernando Miro, Chris Shreeves, Christian Gnoth, Sarah Johnson. Comparison between the different methods developed for determining the onset of the LH surge in urine during the human menstrual cycle. Arch Gynecol Obstet (2015) 292:1153-1161
[2] Direito A1, Bailly S, Mariani A, Ecochard R. Relationships between the luteinizing hormone surge and other characteristics of the menstrual cycle in normally ovulating women. Fertil Steril. 2013 Jan;99 (1):279-85

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Produkt ID: LH-M25

CE 0197

For Self-testing Use
Specimen: Urine
Format: Midstream

INTENDED USE

LH Ovulation Test Midstream is a rapid chromatographic immunoassay for the qualitative detection self-testing of luteinizing hormone (LH) in urine to predict the time of ovulation.

SUMMARY

Ovulation is the release of an egg from the ovary. The egg passes into the fallopian tube where it is ready to be fertilized. In order for pregnancy to occur, the egg must be fertilized by sperm within 24 hours after its release. Immediately prior to ovulation, the body produces a large amount of luteinizing hormone (LH). This is known as the "LH surge" and usually takes place in the middle of the menstrual cycle. LH triggers the release of an egg from the ovary.

LH Ovulation Test Midstream is a complete system to help predict the time of ovulation and peak fertility. It is during this fertile time that pregnancy is most likely to occur.

LH Ovulation Test Midstream detects the LH surge in urine, signaling that ovulation is likely to occur in the next 24-48 hours.

LH Ovulation Test Midstream is a chromatographic immunoassay (CIA) for the rapid qualitative determination of LH in urine. The membrane is pre-coated with anti- α LH capture antibody on the test band region and goat anti-mouse on the control band region. During testing, the urine specimen is allowed to react with the colored conjugate (mouse anti- β LH monoclonal antibody-colloidal gold conjugate) which has been pre-dried on the test strip. The mixture then moves upward on the membrane chromatographically by capillary action. For a positive result, a pink-colored band with the specific antibody-LH-colored conjugate complex will form in the test band region of the membrane, and the color intensity of the test line is as the same as, or stronger than that of the control line. This means the positive results. For a negative result, the color intensity of the test line is weaker than that of the control line. The test result means that the concentration of LH in the urine sample is in the normal level. Regardless of the presence of LH, as the mixture continues to move across the membrane to the immobilized goat anti-mouse, a pink-colored band at the control band region will always appear. The presence of this pink-colored band serves as: 1) verification that sufficient volume is added. 2) That proper flow is obtained, and 3) as a control for the reagents.

COMPOSITION

LH Ovulation Test Midstream contains anti- β LH on a colloid gold particles and a combination of anti- α LH coated on the membrane.

WARNING AND PRECAUTIONS

1. For in vitro diagnostic use only.
2. Do not use beyond the labeled expiration date imprinted on the outside of the foil pouch.
3. Do not reuse the test devices. Discard it in the dustbin after single use.
4. Read directions for use carefully before performing this test. Pay attention to the position of the C and T line.
5. Do not touch the membrane locate in the windows.
6. The instructions must be followed to obtain accurate results.
7. Do not open foil pouch until specimen is collected and ready to be tested.

STORAGE

LH Ovulation Test Midstream should be stored at temperature 2-30°C, the sealed pouch for the duration of the shelf time (36months). Do the test in 1 hour when you open the pouch. DO NOT FREEZE.

SAMPLE COLLECTION

1. Urine sample must be collected in a clean, dry container, either plastic or glass, without preservative. For the best result, collect your urine at about the same time each day, for example, between 10:00 AM and 8:00 PM. Do not collect your first urine after waking up.
2. Samples may be refrigerated (2°C to 8°C) and stored up to 72 hours prior to assaying. If samples are refrigerated, they must be equilibrated to room temperature before testing.

WHEN TO START TESTING


1. Determine the Length of Your Menstrual Cycle.
2. Your Menstrual Cycle Length is the number of days from the first day of your period (menstrual bleeding) to last day before your next period starts. Think back over the last few months to decide what your usual cycle length has been.
3. Circle your usual cycle length on the WHEN TO START CHART below. Select the number directly underneath.
4. Starting the first day of your last period, count ahead the selected number of days on your calendar. This is the day you should begin testing.

Count ahead this many days STARTING WITH THE FIRST DAY of your last period.

Circle Your Usual Cycle Length WHEN TO START CHART	
YOUR CYCLE LENGTH	START TO TEST TIME
21 days	day 6
22 days	day 6
23 days	day 7
24 days	day 7
25 days	day 8
26 days	day 9
27 days	day 10
28 days	day 11
29 days	day 12
30 days	day 13
31 days	day 14
32 days	day 15
33 days	day 16
34 days	day 17
35 days	day 18
36 days	day 19
37 days	day 20
38 days	day 21
39 days	day 22
more than 40 days	Consult doctor

TEST PROCEDURE

Allow the test, urine specimen and/or controls to equilibrate to room temperature (15-30°C) prior to testing. Do not open pouches until ready to perform the assay.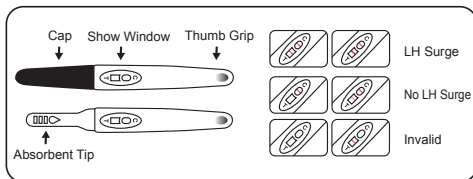

1. Remove the test midstream from the sealed pouch and use it as soon as possible.
2. Remove the Cap to expose the absorbent tip.
3. Hold the midstream by the Thumb Grip with the exposed absorbent tip pointing downward. Urinate on the absorbent tip directly till it is thoroughly wet (at least 5 seconds).
- NOTE:** Be careful do not urinate in the Show Window.
4. Re-cap the test midstream and lay it on a flat surface with window on top.
5. Wait for pink-colored lines to appear. Read result between 3 and 10 minutes. Do not read result after 15 minutes.

INTERPRETATION OF RESULTS

LH Surge: If two pink-color lines are visible and the test line (T) is equal or darker than the control line (C), one will probably ovulate in the next 24-48 hours. If you want to be pregnant, the best time to have intercourse is after 24 hours but before 48 hours.

No LH Surge: Only control line (C) appears or the test line (T) is present but lighter in color intensity than the control line. This indicates that no LH surge has been detected and you should continue daily testing.

Invalid: No line appears in the control region.

NOTE: Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test. If the problem persists, please contact your local distributor.

QUALITY CONTROL

A procedural control is included in the test. A colored line appearing on the control region (C) is considered an internal positive procedural control, indicating proper performance and reactive reagents. A

clear background in the membrane is considered an internal negative procedural control. If the test has been performed correctly and reagents are working properly, the background will clear to give a discernible result.

PERFORMANCE CHARACTERISTICS

Analytic Sensitivity: No less than 25 mIU/ml
Analytic Specificity: The test results show negative for the 200 mIU/ml hFSH and 250 µIU/ml hTSH samples.
Diagnostic sensitivity and Diagnostic specificity: This LH Ovulation Test Midstream has been designed to produce a definitive color band at test region when tested with 25 mIU/ml or higher of LH (WHO 4th IS for human luteinizing hormone) at room temperature. 600 know negative urine samples were equally divided into 6 groups. Each group of samples (100) were spiked with LH to the concentration of 0 mIU/ml, 5 mIU/ml, 15 mIU/ml, 25 mIU/ml, 50 mIU/ml and 5 IU/ml separately. Each group of sample was tested with LH Ovulation Test Midstream. The results from this study gave > 99% agreement with the expected results.

Result	0 mIU/ml	5 mIU/ml	15 mIU/ml	25 mIU/ml	50 mIU/ml	5 IU/ml	Total
Positive	0	0	0	100	100	100	300
Negative	100	100	100	0	0	0	300
Total	100	100	100	100	100	100	600

Diagnostic sensitivity = 100% (300/300)
Diagnostic specificity = 100% (300/300)
Interference Testing: The following substances were added in LH free and LH spiked urine samples. None of the substances at concentration tested interfered in the assay.
For example:

Acetaminophen	0.2 mg/ml	Caffeine	0.2 mg/ml
Acetylsalicylic Acid	0.2 mg/ml	Gentetic Acid	0.2 mg/ml
Ascorbic Acid	0.2 mg/ml	Glucose	20 mg/ml
Atropine	0.2 mg/ml	Hemoglobin	10 µg/ml

LIMITATION

1. Reagents used in the test kit are cross-reactive with HCG. The kit is intended only for the analysis of LH in HCG-free samples.
2. Women suffering form polycystic ovary syndrome may have elevated LH concentration.
3. LH Ovulation Test Midstream will not work properly if you are pregnant, menopausal, or taking birth control pills.
4. LH Ovulation Test Midstream results should not be affected by pain relievers. Antibiotics containing HCG or LH may affect the test and should not be taken while using the ovulation test.
5. LH Ovulation Test Midstream provides a presumptive diagnosis for the time of ovulation. The user should not take any decision of medical relevance without first consulting his or her medical practitioner.
6. The user with eye diseases such as color blindness or color weakness may interpret the result wrongly.

QUESTIONS & ANSWERS

1. Q: Can I use this LH Ovulation Test Midstream to avoid pregnancy?
A: No, the test should not be used as a form of birth control.
2. Q: Do alcohol or common medications affect the test?
A: No, but you should consult your physician if you are taking any hormonal medication. Also, recent oral contraceptive use, breast feeding, or pregnancy could affect the test results.
3. Q: Why should I not use first morning urine? What time of the day should I perform the test?
A: We do not recommend first morning urine because it is concentrated and may give a false positive result. Any other time of day is suitable. For best results, try to collect your urine at approximately the same time each day.
4. Q: Will the amount of liquid I drink affect the result?
A: Heavy intake of fluids prior to testing will dilute the hormone in your urine. We suggest that you limit your fluid intake for about two hours before you perform the test.
5. Q: Once I see a positive result, when is the best time to have intercourse if I want to get pregnant?
A: Once you see a positive result, your ovulation is likely to occur within 24-48 hours. This is your most fertile time. Sexual intercourse after 24 hours but before 48 hours is advised.
6. Q: I received a positive result and had intercourse during these fertile days, but why I didn't get pregnant?

A: There are so many factors that can affect your ability to become pregnant. It often takes normal, healthy couples many months to achieve pregnancy. If you do not get pregnant in the first month, please continue to use the test midstream before achieving pregnancy. If pregnancy is not achieved after 3-4 months, you and your partner should consult your physician.

REFERENCES

[1] Sonya Godbert, Fernando Miro, Chris Shreeves, Christian Gnath, Sarah Johnson. Comparison between the different methods developed for determining the onset of the LH surge in urine during the human menstrual cycle. Arch Gynecol Obstet (2015) 292:1153-1161
[2] Direito A1, Bailly S, Mariani A, Ecochard R. Relationships between the luteinizing hormone surge and other characteristics of the menstrualcycle in normally ovulating women. Fertil Steril. 2013

INDEX OF SYMBOLS

	Do not re-use		Batch code
	In vitro diagnostic medical device		Use-by date
	Store at 2-30°C		Contains sufficient for <n>tests
	Date of manufacture		CE Mark
	Manufacturer		Consult instructions for use
	Authorized representative in Switzerland		